

Quick guide

Patient chair control

Control panel

Touch screen is unlocked. Press to lock. Touch screen is locked. Press and hold to unlock.

Backrest down as long as pressed



Backrest up as long as pressed




Chair down as long as pressed



Chair up as long as pressed



Chair to autoposition

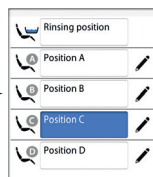


Flexy button

Chair to rinsing position



Foot control

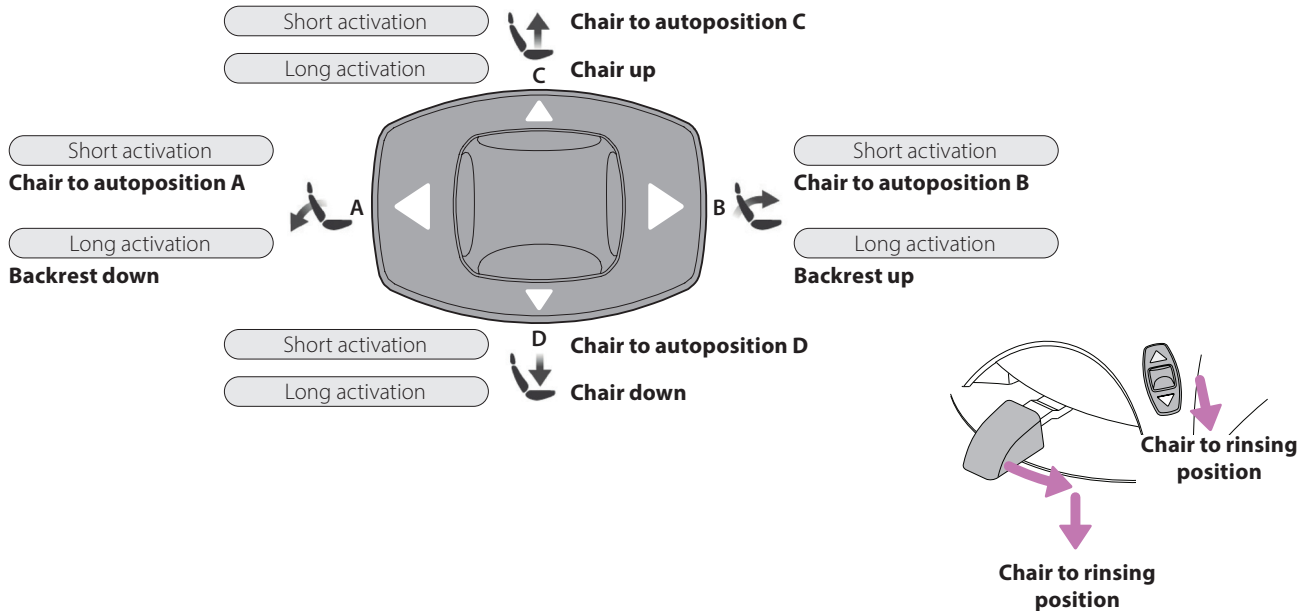

Dental unit control

Control panel



Touch screen is unlocked. Press to lock.



Touch screen is locked. Press and hold to unlock.

Cup fill and bowl rinse

Short activation



Cup fill as long as pressed

Long activation

Bowl rinse

Short activation



Door open / assistant call function for preprogrammed time

Short activation



Door open / assistant call function as long as pressed

Long activation

Operating light On ► Off

Short activation



Intensity of operating light

Long activation

Operating light's composite mode On ► Off

Short activation



Intensity of operating light in composite mode

Long activation

Start required timer



Select timer number

Maintenance procedure



Select procedure

Foot control

No instrument activated

Short activation


Door open / assistant call function for preprogrammed time

Long activation

Door open / assistant call function as long as pressed

Preprogrammed unit function

Preprogrammed unit function

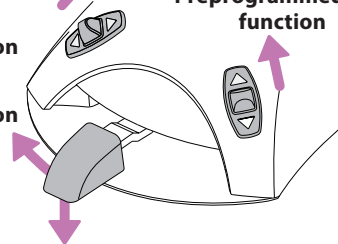

Short activation

Cup fill and bowl rinse

Long activation

Cup fill as long as pressed

Instrument control

Control panel


Touch screen is unlocked. Press to lock. Touch screen is locked. Press and hold to unlock.

Micromotor or turbine Automatic chip blow On ► Off		Micromotor, turbine or scaler Instrument light On ► Off		Micromotor or turbine Water & air ► Air ► No spray Scaler Spray 1 ► Spray 2 ► No spray	
Scaler Change operation mode		Turbine Power reduction On ► Off		Turbine Quickstart On ► Off	
Micromotor Reverse rotation On ► Off		Micromotor RPM limit On ► Off		Micromotors Bien-Air MX2 and Bien-Air MX-i Preset selection	PRESET 4
Micromotors Bien-Air MX2 and Bien-Air MCX Torque limit On ► Off		Micromotor Bien-Air MX2 Toggle drive mode when torque limit On	► ►	Auto stop	Auto reverse

Foot control


Unit controlled instrument activated

Micromotor or turbine (standard pedal)

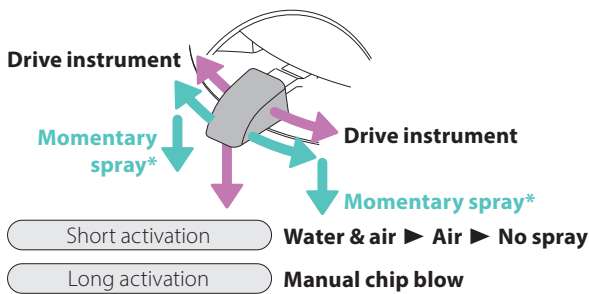


*If enabled by service technician

Scaler (standard pedal)

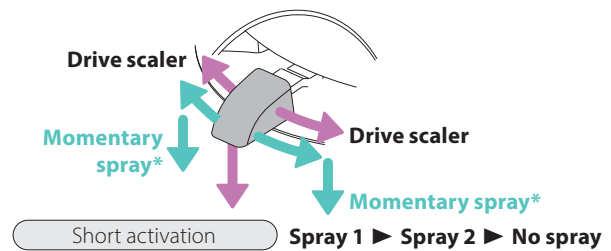


*If enabled by service technician

Micromotor, turbine or scaler

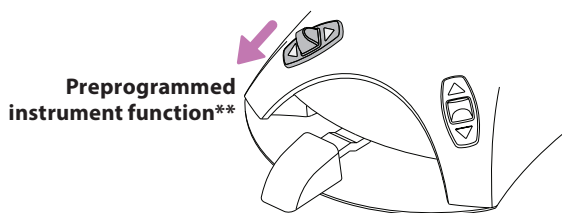


**Can be changed by service technician


Polymerisation light

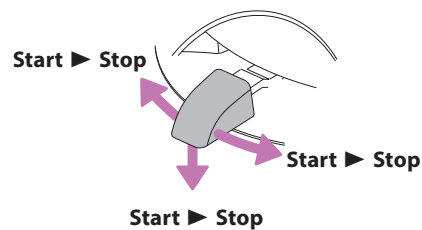

Intraoral camera

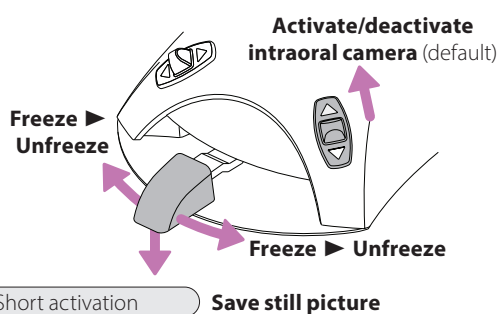


Planmeca intraoral scanner

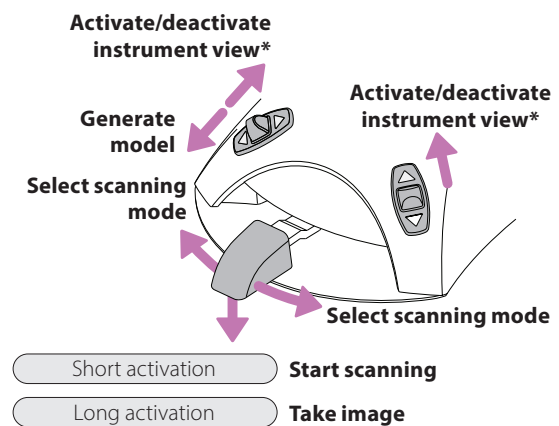


*If enabled by service technician

Motorised headrest control (optional feature)



Foot control


Short activation 


Normal mode
On ► Off


Long activation 


Tilt mode
On ► Off

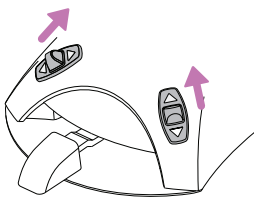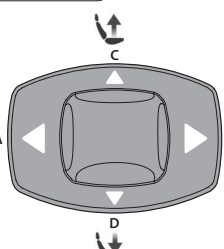




Normal mode  **Lengthen headrest**


Tilt mode  **Move headrest up**


Normal mode  **Anatomical movement down**

Tilt mode  **Tilt headrest backward**

Normal mode  **Anatomical movement up**

Tilt mode  **Tilt headrest forward**

Normal mode  **Shorten headrest**

Tilt mode  **Move headrest down**


Programming

Control panel

 Touch screen is unlocked. Press to lock.  Touch screen is locked. Press and hold to unlock.

Patient chair autoposition settings

 The function is enabled.  The function is disabled.

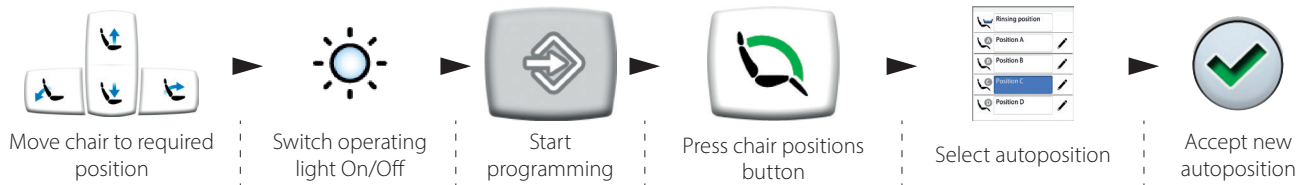

Move chair to required position

Switch operating light On/Off

Start programming

Press chair positions button

Select autoposition

Accept new autoposition

Unit settings

 The function is enabled.  The function is disabled.

Most programming follows this simple sequence:



Start programming

Select function

Adjust value

Accept new value

Instrument settings

 The function is enabled.  The function is disabled.

Most programming follows this simple sequence:



Activate instrument

Start programming


Select function

Adjust value

Accept new value

Time and date settings

 The function is enabled.  The function is disabled.

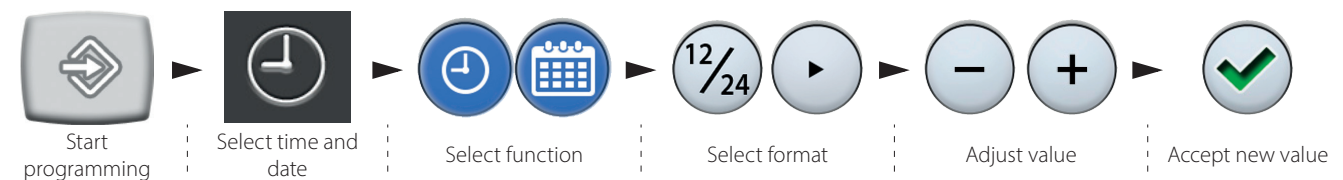

Start programming

Select time and date

Select function

Select format

Adjust value

Accept new value